


**LONGER TREATMENT TIME IS CAUSED BY
A PLUGGED ORIFICE INSIDE THE PARI NEBULIZER.**


**Cleaning & Disinfecting
the Pari Reusable Nebulizer.**

Bring a pot of water up to a boil. Put the Pari LC PLUS, LC Sprint, or LC Star PARTS into the pot of water for 10 minutes.

This should dissolve the medicine clogged in the nebulizer orifice.

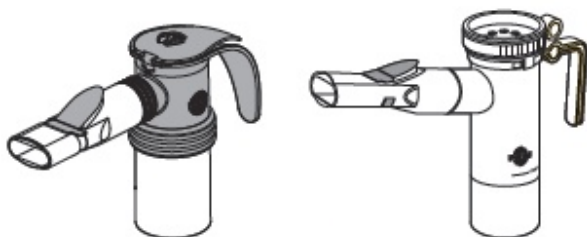

Do Not Put the 7 foot Plastic Tubing in boiling water because it will melt!

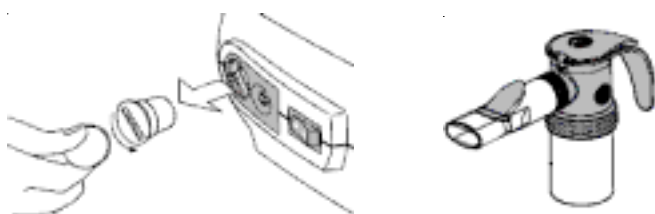

Using a 25 cent coin, Replace the Air Filter on your Vios Compressor every 6 months. Replace the LC Sprint / LC Plus Nebulizer every 6 months.

American Allergy Supply
PO Box 722022 Houston TX 77272
Tel:1-800-221-6483
www.americanallergysupply.com

**The Safest Way To Clean your
Nebulizer with a plugged orifice.**
Use this technique Once a Week!

1. Bring a pot of water up to a boil 210F!
2. Turn the heat off!
3. Put the Pari LC PLUS, LC Sprint, or LC Star PARTS into the pot of water.
4. Come back later to remove the parts from the water and let air dry!!

You will not boil off all the water and melt the nebulizer parts using this technique!

**PARI VIOS 6th Year Warranty
Register your Pari Vios
Compressor Nebulizer!**

Pari allows patients to extend their warranty on the Vios Compressor by "1 year" (6 years total) when they register with
PARI Respiratory Equipment
Call 1-877-943-3931 or
Mail in the Card or Online at

[http://registration.pari.com/
standard-reg-EN.html](http://registration.pari.com/standard-reg-EN.html)